



BLOOD SPILL KIT 5 Ltr

DMRSK05BLD
ABSORBENCY : 5 Ltr

CONTAINS :

One Set of Personal Protective Equipments

- # Safety Goggles : 1 Nos
- # 3Ply MB Mask : 1 Nos
- # Cotton Cargo Apron : 1 Nos
- # Safety Gloves : 1 pair
- # Disposable Shoe Cover : 1 pair

Blotting Paper : 5 Pcs

Disinfectant Liquids : 500 ML

Scoop & Brush : 1 Nos

Barricading tape : 1 Roll

Clinical Wastage Bag : 5 Nos

Blood Neutralizer : 1000 Gms

Caution Mat Yellow : 2 nos

Chemical Absorbent Pads (Green & Blue): 2 Nos

Rejected X-Ray Flim : 2 Nos

KD SOLUTIONS
ONE MOTIVATION COMPLETE SOLUTIONS

Demeanor DMRSK05BLD Blood Spill Kits Specially Custom design and packed for quick & Efficient Response against large large Spillage.

Put on Protective Gears Supplied with Kit.

Sprinkle Granular over the Spill and leave for 2 min.

Once Granules have absorbed the spill, collect same using x-ray sheet & Scoop and brush.

Sanitize the Spill area with disinfectant liquids.

Clean the Spill area using Mats & Pads.

Bag and contents must be disposed of in accordance with local waste disposal policies for biohazard waste.

As per NABH standard and satisfied ISO guidelines.